

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.

**BAUSCH + LOMB**

**re·  
nu**

**CONTACT LENS  
MOISTURE  
DROPS**

**Bausch + Lomb renu**® CONTACT LENS MOISTURE DROPS are a sterile, preserved solution recommended to effectively lubricate and rewet soft (hydrophilic) contact lenses for more comfortable lens wear.

**DIRECTIONS:**

- During wear, put 2 or 3 drops on contact lens whenever needed. Blink several times.
- If the lens still does not feel comfortable, add another drop.
- If discomfort persists, **IMMEDIATELY** remove lens and contact your eye care practitioner.

**CONTENTS:**

A sterile, isotonic solution that contains povidone, boric acid, potassium chloride, sodium borate and sodium chloride; preserved with edetate disodium 0.1% and sorbic acid 0.1%.

**ACTIONS:**

**Bausch + Lomb renu CONTACT LENS MOISTURE DROPS** are ideal for rewetting daily wear or extended wear lenses and disposable lenses or lenses prescribed for frequent replacement. If you wear extended wear lenses, this solution can be used to rewet your lenses after sleep.

**INDICATIONS (Uses):**

**Bausch + Lomb renu CONTACT LENS MOISTURE DROPS** are indicated for use to lubricate and rewet soft (hydrophilic) contact lenses during wear. The product may be used with daily wear or extended wear lenses and with disposable lenses or lenses prescribed for frequent replacement.

**CONTRAINDICATIONS (Reasons not to use):**

Do not use this solution if you are allergic to any ingredient in this product.

**WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE.**

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown that the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care practitioner. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that contact lens wearers see their eye care practitioner twice each year or, if directed, more frequently.

**IMPORTANT SAFETY INFORMATION**

To avoid contamination of your solution or your lenses:

- Do not touch tip of container to any surface.
- Replace cap after using.

Do not transfer this solution to other bottles or containers.

**PRECAUTIONS:**

- Always clean, disinfect, rinse and soak your lenses each time they are removed.
- Keep the bottle tightly closed when not in use.
- Store solution at room temperature.
- Use before the expiration date marked on the carton and bottle.
- **Keep out of reach of children.**
- Discard any remaining solution within 90 days after opening.

**ADVERSE REACTIONS (Problems and what to do):**

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above:

**Immediately remove your lenses.**

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, **DO NOT** put the lens back on your eye. Place the lens in the lens case and contact your eye care practitioner.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens, then reinsert it.
- If the problem continues, **IMMEDIATELY** remove the lens and consult your eye care practitioner.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate practitioner identification of the problem and prompt treatment to avoid serious eye damage.

All adverse reactions observed while using **Bausch + Lomb renu**

**CONTACT LENS MOISTURE DROPS** should be reported to:

Bausch & Lomb, Rochester, NY 14609, 1-800-553-5340.

**GOOD LENS CARE PRACTICES:**

- Always wash and rinse your hands before you handle your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens first, the right or the left, to avoid mix-ups.
- Before first and after each additional use, always empty and rinse lens case and caps with fresh rinsing solution and allow to air dry
- When wearing disposable lenses, each time you remove a lens, discard it and replace it with a new lens.

**HOW SUPPLIED:**

**Bausch + Lomb renu CONTACT LENS MOISTURE DROPS** are available for purchase in 0.27 FL OZ (8 mL) plastic bottles. Bottles and cartons are marked with a lot number and expiration date.

**Toll Free Product Information**

**Call: 1-800-553-5340**

®/TM are trademarks of Bausch & Lomb Incorporated or its affiliates.

© 2020 Bausch & Lomb Incorporated or its affiliates

**Manufactured by:**

Bausch & Lomb Incorporated  
1400 North Goodman Street  
Rochester, NY 14609  
www.renu.com  
www.bausch.com